
MICROPUMP SYSTEM
CORRECTIONS TO THE USER'S MANUAL

Important Safety Information / Wichtiger Sicherheitshinweis / Informazioni importanti sur la sécurité / Informazioni di sicurezza importanti / Información importante de seguridad / Informações importantes de segurança / Informaçoões importantes de segurança / Belangrijke veiligheidsinformatie / Viktig sikkerhedsoplysninger / Viktig informasjon om sikkerhet / Viktig säkerhetsinformation / Tärkeä turvallisuuettietoja / Ważne informacje dotyczące bezpieczeństwa / Důležitá bezpečnostní informace / معلومات هامة للسلامة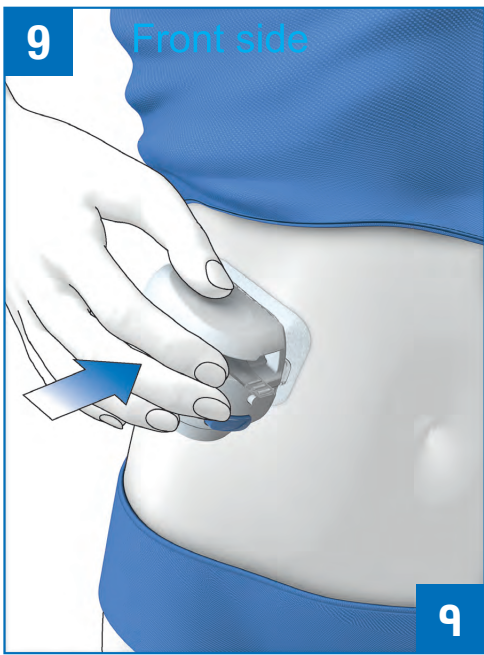

EN Corrections to the User's Manual

Chapter 4.1 Recommended Infusion Sites

The note correctly reads as follows:

NOTE

- Do not attach the pump holder to body sites with scars, birthmarks, moles, tattoos, injuries, bruises, or rashes.
- Choose a flat infusion site that is not too curved so that the adhesive pad of the pump holder adheres well.
- Choose an infusion site that is easy to reach and see.
- A new infusion site must be at least 5 cm away from the last infusion site.
- Before you attach the pump holder to the body, the infusion site must be completely dry.
- Do not touch the introducer needle or the cannula of the cannula assembly.
- If the pump holder frequently detaches from the skin, consult your healthcare professional to find a method that may improve adhesion to the skin.
- If the infusion site becomes inflamed or localised skin reactions occur (for example, allergic reaction or eczema), then replace the infusion assembly immediately and choose a new infusion site.

Chapter 4.2.1 Attaching the Infusion Assembly to the Body

The new warning and steps 9 and 10 correctly read as follows:

▲ WARNING

Risk of hyperglycaemia (high glucose level)
Improper handling of the insertion device may lead to leakage. This may lead to hyperglycaemia.

Carefully follow the instructions for using the insertion device as described in the steps below.

- █ Holding the skin taut, press the insertion device firmly against the selected infusion site without tilting, twisting or wiggling. Apply force only in the direction of the infusion site as shown in the illustration.
- █ While holding the insertion device firmly in place, press the blue release button to insert the cannula under the skin.

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Chapter 4.2.6 Attaching the Micropump

The warning, note and step correctly read as follows:

▲ WARNING

Risk of hyperglycaemia (high glucose level)
• Deformations, cracks, or frequent and incorrect insertion of the micropump into the infusion assembly may cause the micropump system to become leaky. Before attaching the micropump to the infusion assembly, check the micropump and the pump holder for damage.
• Take special care not to bend the reservoir needle while attaching the micropump to the infusion assembly. A bent reservoir needle can cause the micropump system to become leaky.

NOTE

Check the site on your body with the adhesive pad of the infusion assembly at least once a day.

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NOTE

Check the site on your body with the adhesive pad of the infusion assembly at least once a day.

▲ WARNING

Risk of hyperglycaemia (high glucose level)
Improper handling of the insertion device may lead to leakage. This may lead to hyperglycaemia.

Carefully follow the instructions for using the insertion device as described in the steps below.

- █ Holding the skin taut, press the insertion device firmly against the selected infusion site without tilting, twisting or wiggling. Apply force only in the direction of the infusion site as shown in the illustration.
- █ While holding the insertion device firmly in place, press the blue release button to insert the cannula under the skin.

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